JYSK HANDI

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| Button | Setting | LED | Funktion |
|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-------------|------------------------------------------------------------------------------------------------------------------------------------------------------|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
|    | |  |  |
|  | Holdtime | <ul style="list-style-type: none"> • blue 0,5s • green 1,0s • red 1,5s • yellow 2,0s ○ white 2,5s | <p><i>AS one in Tonemode:</i> Span of time the sound must be kept inside the set frequency range</p> <p><i>AS one is Impulsemode:</i> Button has no function when AS one is set to impulsmode.</p> |
|  | Sensitivity | <ul style="list-style-type: none"> • blue high • green • red • yellow ○ white low | When set to high sensitivity, AS one reacts to weak input signals. |
|  | Tone range | <ul style="list-style-type: none"> • blue wide • green • red • yellow ○ white narrow | <p><i>AS one in Tonemode:</i> Accepted deviation from the frequency presetting.</p> <p><i>AS one is Impulsemode:</i> Button has no function when AS one is set to impulsmode.</p> |

Change settings

By repeated pressing of one button, you can move through the whole adjustment range. The LED shows the actual setting. One second after the button is released, the LED starts blinking for two seconds.

| | | |
|-----------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------|
| Enter setting | Press the button while the LED is blinking to save the new setting. AS one confirms with a longer sound. |  |
| Discard changes | Leave the button released until the blinking LED turns off after two seconds AS one keeps on using the last saved setting and confirms with the buzzer sounding short. |  |

4. Tekniske data

| Beskrivelse | |
|--------------------|-----------------------------|
| Powersupply | 12VDC via AssistX Port |
| Connection | AssistX Port. 3.5mm jack |
| Range of Frequency | 80 - 600 Hz (Tone tilstand) |

| | |
|-------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| DIN VDE 0834 | AS one together with AssistX Call or AssistX Mobil meets the requirements of DIN VDE 0834 for call systems. |
| Interval of inspection | Within the scope of DIN VDE 0834 for call systems, AS one has to be inspected four times a year. |
| Replacement cable | Be shure to always have a replacement cable to connect AS one to an AssistX Port |
| Cleaning | AS one may be damp cleaned using a mild detergent. Don't dip into water! |
|  | <p>The mounting plate contains a strong neodym magnet. Magnets can affect cardiac pacemakers and implanted defibrillators.</p> <ul style="list-style-type: none"> • The pacemaker could be switched into testmode and cause discomfort. • A defibrillator could stop working. • Keep enough distance to magnets if you use such devices. • Advice other users of such device against getting too close to magnets. <p>Recommended safty distance is 4 inch (10 cm)</p> |
|  | <p>Magnets cause a far-reaching magnetic field. They could damage TV sets, laptops, PC harddrives, creditcards, ATM cards, data storage, mechanical watches, hearing aids and speakers.</p> <p>Vehicle keys, USB sticks, cameras, cell phones and smart phones contain non-magnetic storage media. Therefore, static magnetic fields near those devices cannot delete data.</p> <p>There are countless covers with magnetic closures or magnetic holders on the market, which also speaks against the notion that magnets can damage these devices.</p> <p>However, it cannot be ruled out that very strong magnetic fields might magnetise and maybe damage mechanical parts or built-in speakers.</p> <p>When in doubt, keep these devices away from strong magnets.</p> |

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CSS Microsystems GmbH